


DIGI WAXER

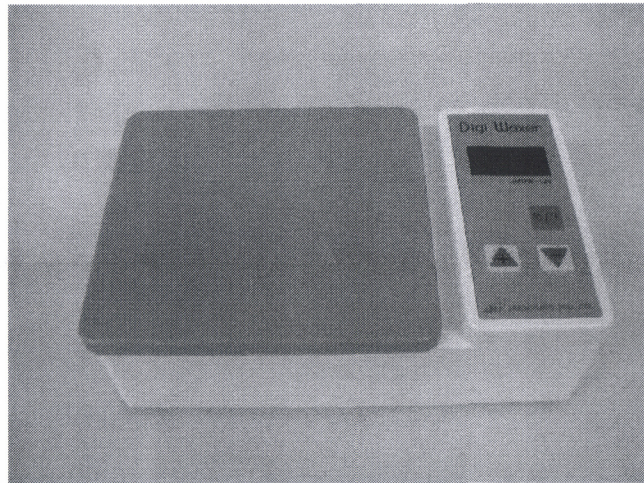

USER'S MANUAL

A.M.D Dental Mfg




1. Caution

Don't touch the inside wax pot with hands during your operation..

2. How To Use

- 1) Please connect power cord.
- 2) Please Turn on the Main switch
- 3) Please choose °C or °F.
- 4) When you press (+) or (-) one time, the display shows an input temperature. Please memorize a temperature you want with (+) or (-)
- 5) A few seconds later, the display show a current temperature.

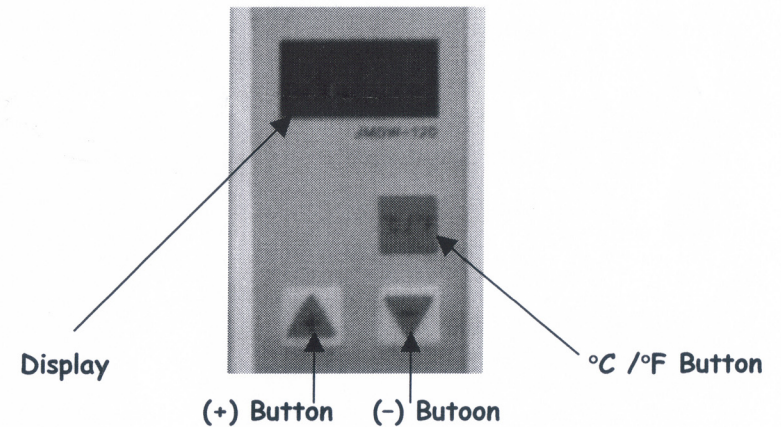

3. Specification

Power	115V, 60Hz, AC2.5A
Consumption	330W(Max)
Size	(W)185 × (D)115 × (H)63
Max Temp	120°C

Version 1.0 dated of June 12, 2005