



A.M.D. DENTAL MFG., INC.

318Cleveland Ave. Highland Park, NJ 08904
TEL : (732)846-2288 OR (732)249-3332 OR
TEL : (732)846-1299 FAX : (732)249-1288
WEBPAGE : www.amddental.com E-MAIL : HANNA1205@AOL.com

Surveyor AMD-105:

Instructions:

1. When using the handpiece, you must attach the black handpiece holder to the end of the flexible arm. The holder has two holes. The one with the open is used to secure the holder to the pole at the end of the flexible arm; the other hole is for the handpiece. The two handles on either side are used to tighten the grip on either the center pole or the handpiece.
2. The seven spindles are located underneath the black case on the very top.
3. Put object on top of vise and secure by tightening the top screw; use the bottom screw to change the angle of the vise, from 0-45 degrees.
4. Flexible arm can be adjusted to move vertically by turning the screw on the central pole. The other screw right above is used to tighten or loosen the flexible arm's position as desired.
5. Pole at the end of the flexible arm can be fixed into position as that it will not move; in order to do this one must turn the handle located right above the spindle holder.
6. In order to place the spindles, the knob at the end of the flexible arm must be loosened until the spindle will fit into the bottom hole. Then tighten the knob to secure the spindle.
7. To lock the vise onto platform, put the vise over the round point on the center of the platform; then tighten the screw on the side of the platform to lock into place.

